USER MANUAL (2 - 15)
REMOTE CONTROL 2.0
FOR HEARING SYSTEMS

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www.audioservice.com

Dear customer

We are delighted that you have chosen a hearing system with remote control from Audio Service.

We are committed to supporting you with all your hearing and comprehension needs. For over 30 years, Audio Service has been developing hearing systems at its Herford site and supplying the market with cutting-edge, high-quality products that offer outstanding support in all hearing situations.

This user manual will help to familiarise you with the functions and applications of your remote control.

If you require further information, please contact your hearing system specialist who will be happy to help.


INTENDED USE

The intended use of hearing systems is to improve the hearing of people with hearing difficulties. The remote control is intended to control hearing systems. Please note that the diagnosis and prescription of hearing systems must be performed by a qualified person e.g. an ear, nose and throat specialist, audiologist or hearing system specialist.

Please read and follow the instructions in this user manual in order to prevent injuries and protect your hearing systems and accessories from damage.

YOUR REMOTE CONTROL

The remote control requires two AAAA batteries (for further information on this see p. 9).

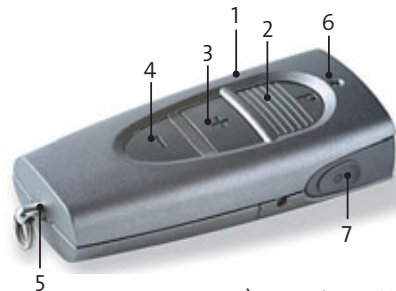- | | |
|----------------------------|---|
| 1) Keylock | 5) Keyring attachment |
| 2) Programme button | 6) LED |
| 3) Volume control "louder" | 7) Switch hearing systems on/off (stand-by) |
| 4) Volume control "softer" | |

LED

LED lights up once when a button is pressed. It does not light up when the keylock ① is activated.

KEYLOCK

Keylock on

Keylock off, green area visible.

Your remote control has a keylock (see ①, page 3). When it is activated, all other buttons are deactivated to prevent accidental operation.

ACTIVATING THE KEYLOCK

Push the switch to the right to activate the keylock (see Fig. "Keylock on")

RELEASING THE KEYLOCK

Push the switch to the left to deactivate the keylock (see Fig. "Keylock off")

SWITCH HEARING SYSTEMS ON/OFF (STAND-BY)

Press the On/Off button to switch on your hearing system. Press the On/Off button to switch off your hearing system (stand-by).

FUNCTIONS

Your remote control allows you to change the volume or the hearing programme for both hearing systems by pressing just one button.

NB:

- If the settings of your hearing system are changed by another person's remote control, please inform your hearing system specialist. He or she can send in your remote control to have the radio frequency changed.

NOTE ON USING AUDIO INPUTS:

- If you use one or two hearing systems with an audioshoe, you can control the volume with your remote control as described.
- The programme button is used to change the audio mode when the audioshoe is attached. More information on this can be obtained from your hearing system specialist.

USING THE VOLUME CONTROL

You can use the **volume buttons** “+” and “-” to change the volume of your hearing systems.

Use the **“+” volume button** to increase the volume.

Use the **“-” volume button** to decrease the volume.

IMPORTANT:

■ **If you wear two hearing systems, the volume changes in both hearing systems.**


Your hearing system specialist can set the hearing systems for you so that you hear a signal when the volume is changed. When the maximum volume is reached, you will hear several short signals.

USING THE PROGRAMME BUTTON

You can change the programmes of your hearing systems using the programme button (see Fig. left). The programmes are sequentially arranged. When you switch off your hearing systems or change the batteries, the hearing systems will revert to programme 1 when you switch on again. Pressing the programme button always switches to the next programme. The sequence is in ascending order. After the last available programme, the sequence begins again with the first programme.

Your hearing system specialist will set the hearing programmes for you according to your personal needs and acoustic requirements. Your hearing system can be set to give an acoustic signal when switching programmes. One signal is heard when you switch to the first programme. When you switch to the second programme, two signals are heard, and so on.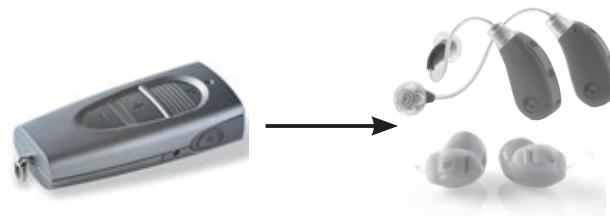

Maximum distance in order to change programmes or volume: **1 metre**

IMPORTANT:

■ If you wear two hearing systems, the programme changes in both hearing systems when you switch programmes.

In unfavourable conditions (bad reception), differences may occur in the settings of the right and left hearing systems. In this case, hold down the “P” button on the remote control for at least 3 seconds (reset). This resets both hearing systems to programme 1.


YOUR PERSONAL HEARING PROGRAMME SETTINGS:

Hearing programme *	Description of setting
1	
2	
3	
4	
5	

* There are 5 possible hearing programmes, depending on the hearing system and the settings performed by your hearing system specialist.


CHANGING THE BATTERIES

The remote control uses two AAAA batteries.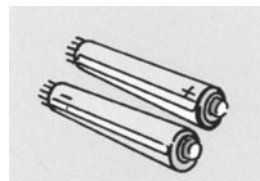

AAAA batteries

To change the batteries, open the battery compartment by pulling it downwards. Insert the batteries. Note the polarity symbols (+ and -) displayed in the battery compartment.

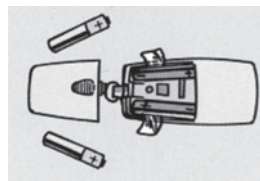Changing the batteries

Always use fresh batteries. Do not mix used batteries with fresh ones.

Take your old batteries to your hearing system specialist for correct disposal.

INSTRUCTIONS ON USING THE REMOTE CONTROL

- You can use your remote control completely inconspicuously from your trouser pocket or handbag.
- To return your hearing system to its basic setting (preset volume control and programme 1), hold down the programme button for 3 seconds (= reset).
- When near to water, use the protective cover to protect your remote control from splashing. Ensure that your remote control does not come into contact with water. The protective cover does not make your remote control watertight.
- Do not clean your remote control with alcohol or benzine. The coating may become damaged over time. Clean the remote control with a soft cloth. Do not use aggressive cleaning agents such as citric acid, ammonia, scouring agents etc..

SAFETY INSTRUCTIONS

- Store batteries well out of reach of babies and small children as there is a danger of swallowing and suffocation. Consult a doctor or hospital immediately if batteries or other accessories are swallowed.
- If you are not going to use your remote control for a long time, remove the batteries. Batteries can leak. If this happens, remove them immediately.
- The remote control is sensitive to extreme heat, high humidity, strong magnetic fields (>0.1 T) and mechanical stress. Never expose your remote control to extreme temperatures or high humidity. Avoid direct sunlight. Do not use your remote control if you are exposed to short waves, strong magnetic fields or X-rays.
- The performance of the wireless function may be impaired by sources of electromagnetic interference – e.g. by monitors or fluorescent tubes. Move away from these sources of interference if disruptions occur.

Wireless systems can cause disruptions to measuring instruments and electronic devices. Do not use the remote control in hospitals or aeroplanes. Activate the keylock in hospitals and aeroplanes. Before using the remote control in the vicinity of an electronic implant or life-support system, have the electromagnetic compatibility tested.

- Tests have confirmed that the remote control does not cause any demonstrable interference with pacemakers that are manufactured and operated in accordance with the applicable safety standards. The remote control can therefore be used without problems by people who have these pacemakers or in their proximity. If you have a pacemaker, keep the remote control at a safety distance of 30 cm away from the pacemaker. Do not carry the remote control:
 - » in your breast pocket
 - » on a cord around your neck
 - » directly against the skin above the implanted pacemaker
- As with all electrical, non-gas-encapsulated devices, you must not use your remote control in areas where there is a risk of explosion.

- Some countries impose certain restrictions on the use of radio devices. More information can be obtained from the responsible local authorities.

- When the hearing system is used in T-induction mode, the wireless function may generate a pulsating sound. This is caused by the T-induction loop of the hearing system picking up the transmission signal. When your hearing system is in T-induction mode, please use the remote control only at a distance of at least 10 cm.

Do not dispose of electrical devices in the household waste! Please send the remote control back to the manufacturer for disposal (address see back page).

TECHNICAL DATA

Frequency range:	3.28 MHz
Maximum field strength:	-7 dB μ A/m at a distance of 10 metres
Operating voltage:	3.0 V
Batteries:	2 x batteries 1.5 V, type AAAA

TROUBLESHOOTING

Problem	Solution
If the remote control does not work:	<p>Check whether the key-lock is activated. If it is, deactivate it.</p> <p>Move away from electromagnetic sources.</p> <p>Change the batteries.</p>
Your hearing system is affected by another remote control:	All remote controls work on the same radio frequency. It is highly unlikely that another user's remote control will affect your hearing system within a one metre range. If this should occur, however, please inform your hearing system specialist. He or she can send in your remote control to have the radio frequency changed.

ACCESSORIES

Your remote control has an attachment for a keyring, purse or wallet. Other accessories:

**Clip**

You can use the clip to fix your remote control to your belt.

**Protective cover**

The protective cover protects your remote control from water splashes and scratches.

NB:

■ The protective cover does not make your remote control watertight. Never allow your remote control to get wet.

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Audio Service GmbH • Zeppelinstraße 9 • D-32051 Herford